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TUMOR MARKING NEEDLE

INSTRUCTIONS FOR USE:

weLLgo Tumor Marking needle is a single lumen needle with ergonomic handle and with pre-loaded fiducial gold marker at distal tip.

Indications For Use: It is a needle for placement of tumor marker before radiation treatment and/or pre-surgery localization of soft tissue tumors like; Prostate, Breast, Liver and Lung.

Tumor markers are implanted in the tumor tissue to act as a radiologic landmark for ensuring accurate target localization of tumors during radiation therapy.

Contraindications: Biopsy / aspiration sets or needles should be used by a physician trained in interventional techniques and familiar with the possible side effects, typical findings, limitations, indications and contraindications of biopsy procedures. Physician judgement is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

Possible Side Effects: Pain, fever, bleeding, hematuria and infection.

<u>Caution:</u> Ethylen oxide sterilized. Non-pyrogenic. Check the package if package damaged or opened do not use the products. Check the expiry date and the gauge. Store in dry place, under 5-30°C temperature, protect from sun light. Possible allergic reactions should be considered. It must be used only in hospitals by physicians who familiar with the device and procedures. For single Patient Use Only. Do not attempt to clean or resterilize this product. Do not use the device under MRI. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

LOCALIZATION PROCEDURE:

- Identify the lokalization site preferably under Ultrasound, X-Ray or CT.
- 2. Using the sterile technique, prepare the skin with antiseptic and drape.
- 3. If necessary, infiltrate the selected area with local anaesthetic.
- 4. For prostate seeding, place the Transrectal probe to right position.
- Open the needle pouch and remove protective sheath from the needle.
- 6. Insert the Tumor marking needle (for prostate seeding through Ultrasound probe) and advance into the targeted area.
- 7. Set the Tumor marking needle in the lesion.
- 8. Place tumor markers into lesion as shown in Fig.1 and Fig.2
- 9. Remove the needle.

<u>Note:</u> These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

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
 Documentation No:
 Valid from:
 Revision no:
 Revision date:


Abb. 1 Exposing and placing tumor marker by pushing fordward the handle part

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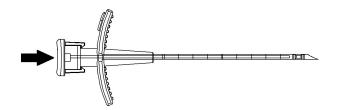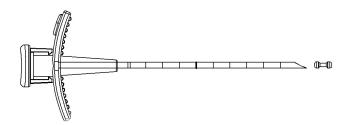weLLgo Medical Products GmbH Buchenhofener Str. 21, 42329 Wuppertal, GERMANY